**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

**INTENDED USE/INDICATIONS:** Support for Patients with Unicompartmental Degenerative Joint Disease. This product was designed to complement a variety of medical treatments common to the above afflictions. The frequency and duration of use should be determined by your prescribing physician.

**CONTRAINDICATIONS:** Please be advised that dj Orthopedics, LLC does not recommend simultaneous wearing of rigid knee braces bilaterally (right and left legs), during activities (ie skiing) that may cause the braces to contact each other, resulting in potential loss of control and injury. Rigid knee braces are not recommended to be worn over clothing. Consult your licensed Healthcare professional for your activity limitations.

**WARNINGS AND PRECAUTIONS:** Always consult with your physician or therapist before making changes to the brace. This product is to be used under the supervision of a medical professional. This device is not intended for unsupervised public use. If you experience any pain, swelling, sensation changes or any unusual reaction while using this product, consult your medical professional.

**APPLICATION INFORMATION:**

- 1) While sitting on the edge of a chair, flex the knee to 45 degrees. Position the brace as shown in figure 1, centering the hinges approximately 1" (2.54cm) above the top of the knee cap and slightly behind midline of the leg. Make sure the brace is not twisted on the leg. (Fig. 1)
- 2) First wrap strap #1 (upper calf) behind the upper calf. Secure the buckle on the strap firmly into the buckle mounted on the frame. Loosen the strap end and pull snug. Be sure not to over tighten any of the straps as this may cause reduced circulation. Secure strap #2 (lower calf) by wrapping the strap behind the lower calf. Secure and adjust the straps as instructed above. Continue with strap #3 (upper thigh) and strap #4 (lower thigh). Secure and adjust the straps as instructed above. The anterior tibial (front shin) strap can be adjusted upon initial fitting to be snug against the shin. It will not require retightening with each fitting. **Note:** Be sure to maintain proper hinge alignment on the knee during the strapping of brace. If necessary, push the hinges back on the knee to slightly beyond midline of the leg. (Fig. 2)
- 3) When all strapping is complete, walk for a few minutes and then retighten all straps following the same sequence as above. The hinge may move slightly lower to find the anatomical center of the knee. (Fig. 3)
- 4) To remove the brace push the quick release buckles located on each strap and remove the brace. (Fig. 4)

**INSTRUCTIONS FOR ADJUSTABLE LOADING HINGE:**

To achieve the proper amount of corrective force to alleviate the symptomatic pain associated with unicompartmental osteoarthritis, the OA Everyday Brace must be adjusted to the patient by a trained medical professional. You may "fine tune" your base load by doing the following: Turn the adjustment screws clockwise using the enclosed Allen wrench tool. You may vary the front and back screw adjustment depending on its interface with the knee and comfort of load. DO NOT turn any screw more than 3 revolutions without toggling back to the other adjustable screw. Be careful not to bind the adjustment mechanism. Do not exceed 20 full revolutions or the pad may become dislodged. If this occurs, reverse the process. Always load brace according to your comfort and activity level. **NOTE:** Even if you don't feel immediate relief, it doesn't mean the brace isn't working. It can take several weeks for soft tissue in the knee to adjust to the increased load provided by the brace.

**INSTRUCTIONS FOR EXTENSION/FLEXION STOPS:**

**Standard Hinge** - To install stops, first remove the condyle pad, then remove the screw as shown. Insert the stops, align with the holes in the hinge plate, then reinsert the screw. Using a Phillips head screwdriver, ensure that the screw is seated in the recess and completely tightened. (Fig. 5)

**Adjustable Hinge** - To change your stops on the adjustable loading hinge, you must first remove the adjustable loading condyle from the hinge by toggling back and forth turning the screws clockwise until the pad is removed. Remove the stop and insert the new stop. To replace the telescoping pad, align the screws with the holes of the hinge plate. While holding it firmly, use the Allen wrench to turn counterclockwise until the threads engage. Toggle back and forth between screws until the pad is returned to its original position. (Fig. 6)

Both medial and lateral stops must be identical in degree.  
Extension Stops: 0°, 10°, 20°, 30°, and 40°. Flexion Stops: 0°, 45°, 60°, 75°, and 90° (no stops are installed upon shipment).

**INSTRUCTIONS FOR TIBIAL CREST PADDING:**

The black (adhesive back) foam is to be used for shin or tibial crest discomfort. Cut the foam to the appropriate height of the cuff. Note the area of discomfort where the leg meets the cuff. Remove the cuff padding and adhere a piece of foam on each side of tibial crest, saddling the crest. You may find it necessary to mutilate the foam to achieve the correct depth. Replace the cuff padding. If discomfort persists, discontinue use and notify your physician.

**INSTRUCTIONS FOR CONDYLE PAD KIT:**

Your brace has a condyle padding kit that contains 2, 3 & 5mm condyle pads. These pads should be used to take up space on the medial or lateral side. They can be multi-layered.

**USE AND CARE:**

All straps and pads may be trimmed to fit individual variations in limb circumference. The brace may be used in fresh or salt water. Following use in water activities, drain thoroughly and rinse with clean, fresh water then air dry.

**CLEANING INSTRUCTIONS:** To clean liners, hand wash in cold water using mild soap. Air dry. Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

**WARRANTY:** Congratulations on your purchase of a DonJoy® brand brace. Please be assured that it is made of the finest material and technology available. Your brace comes with a 6 month warranty against defects on soft goods associated with the brace. The OA Everyday brace has a 1 year warranty (off-the-shelf brace) or a limited lifetime warranty (custom brace) against defects on material and workmanship for the brace frame and hinges. Replacement straps, liners, ect., can be purchased from our local representative, medical provider, or by calling Customer Care at (800) 336-6569, fax (800) 936-6569.

Patent pending.  
For single patient use only.

**NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THIS DEVICE IS ONLY ONE ELEMENT IN THE OVERALL TREATMENT PROGRAM ADMINISTERED BY A MEDICAL PROFESSIONAL. THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.**

**CAUTION: FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PROFESSIONAL.**

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